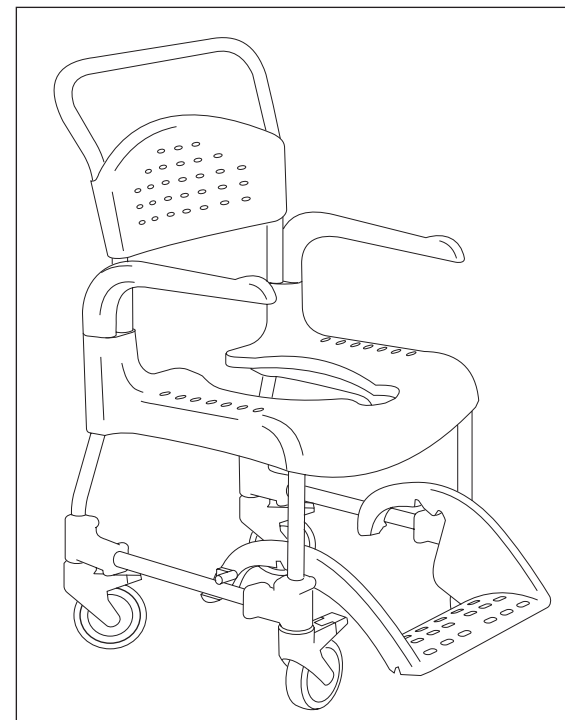
Clean

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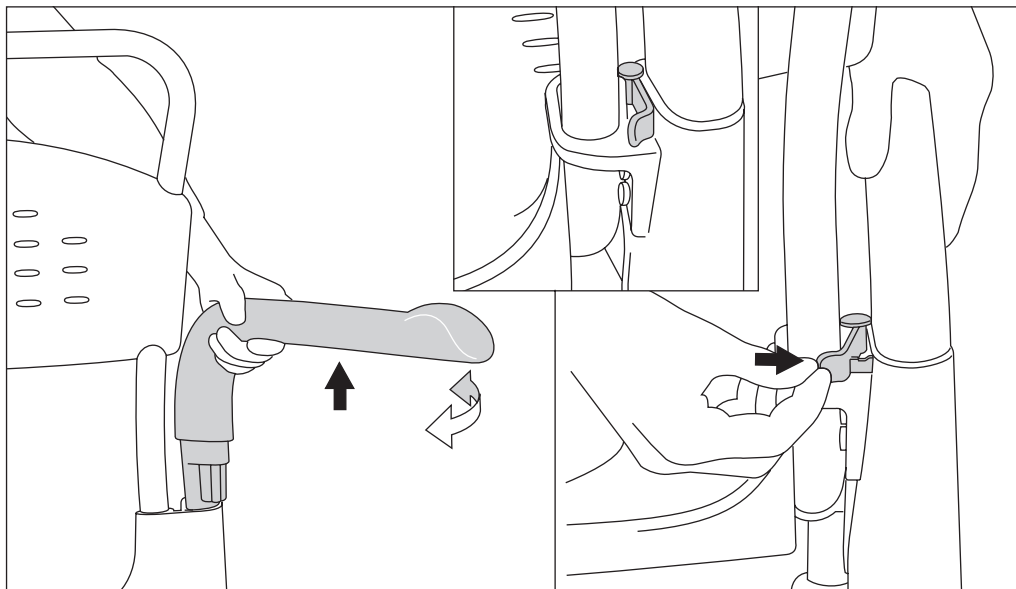
(Eng) Mobile shower and toilet chair which simplifies personal hygiene for persons with impaired mobility and their carers.

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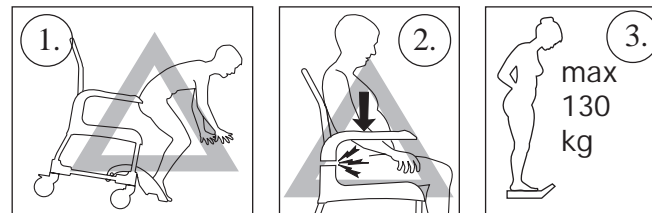
(Eng) Armrest
The armrests are swing away and detachable.
The armrests are lockable.

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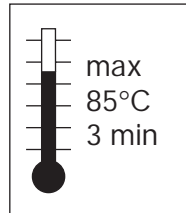
(Eng) Symbols
1. Never stand on the footrest as the chair may tip over.
2. Caution: risk for pinching at armrest joint.
3. Maximum user weight: 130 kg.

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(Eng) Symbols

Clean the chair with ordinary non-abrasive detergent. Rinse and dry.

The product can be cleaned at 85° for 3 minutes.

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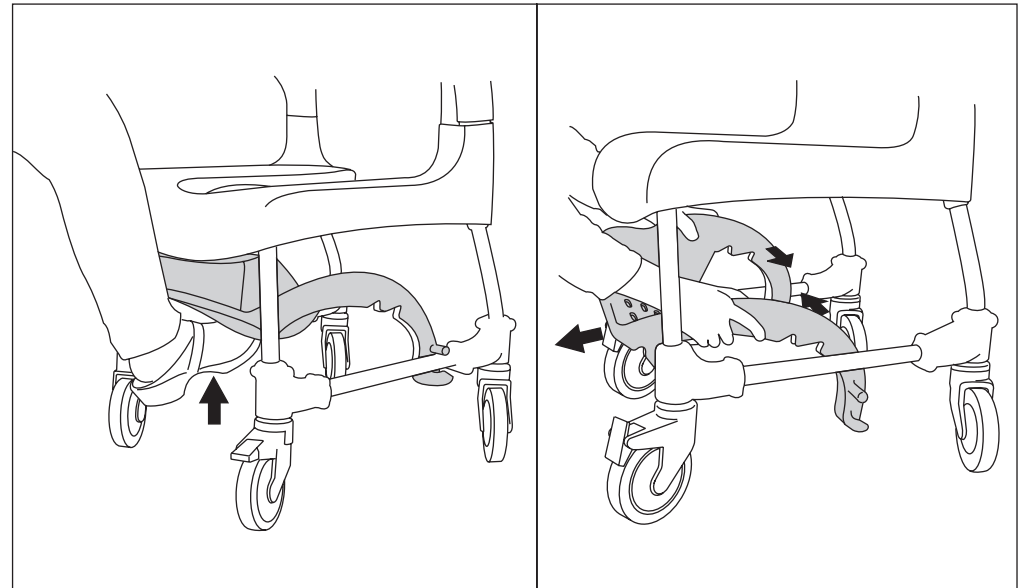
(Eng) Footrest

The carer can easily push in the footrest with their foot. The footrest is detachable.

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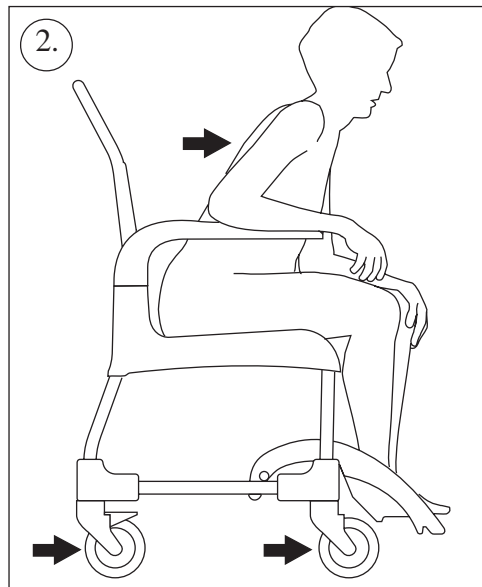
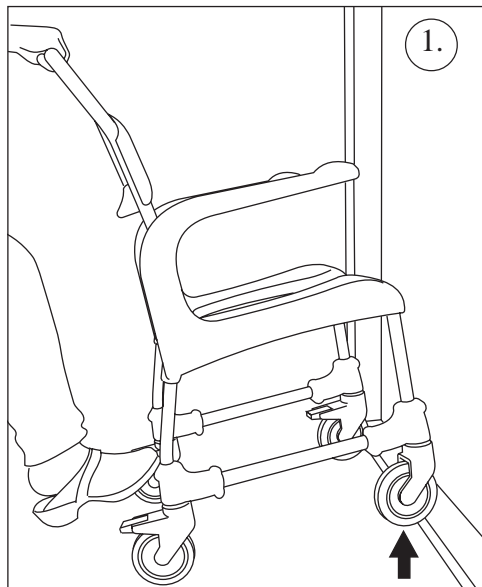
(Eng) 1. **Over a threshold:** Chair only, push down on the wheel attachment. With occupant, pull the chair backwards.

2. **In stationary position:** Turn the chair so the castors are pointing forwards.

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(Eng) Maintenance and safety

Product lifetime:

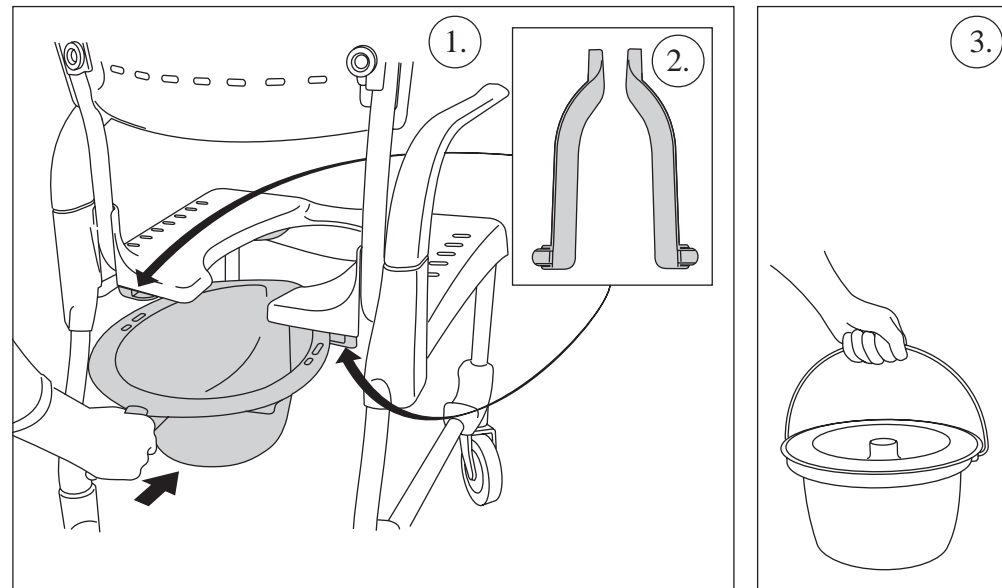
10 years (excluding soft seat/back) providing that Clean is cleaned and maintained.

Maintenance:

To keep the chair in good and safe working order, you should regularly:

- Clean the chair with ordinary non-abrasive detergent. Rinse and dry.
- Keep the castors clear of hair and soap deposits, etc.

If your shower and toilet chair is not in perfect working order, contact your dealer. A defective chair should not be used.

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(Eng) Accessories

1. The pan slides in from the rear.
2. Pan holders.
3. Bucket holders are required for attaching a bucket.

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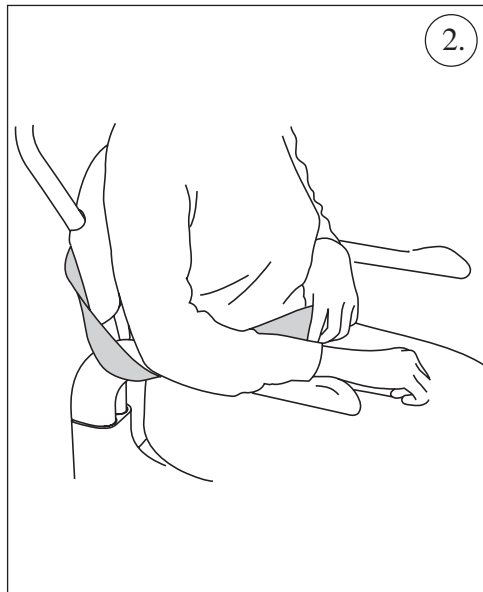
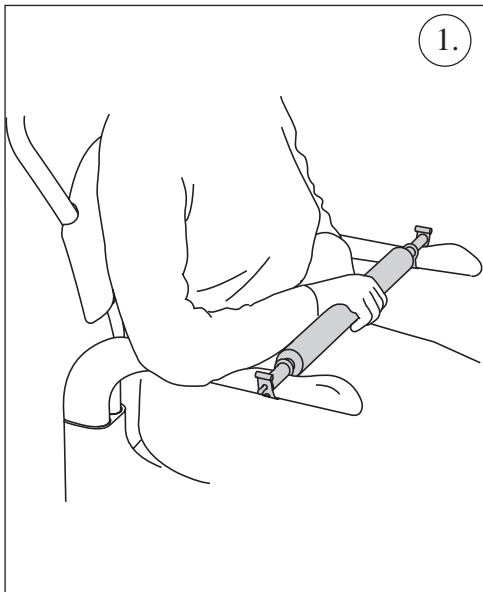
(Eng) Accessories

1. The cross bar is fitted to the armrests.
2. The safety belt is attached to the rear of the backrest, and is pulled forwards under the armrests.

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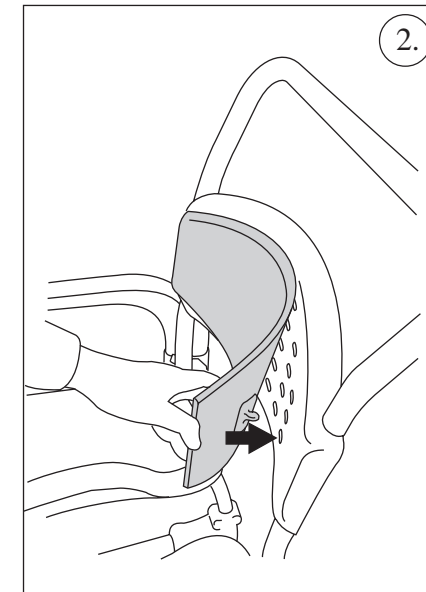
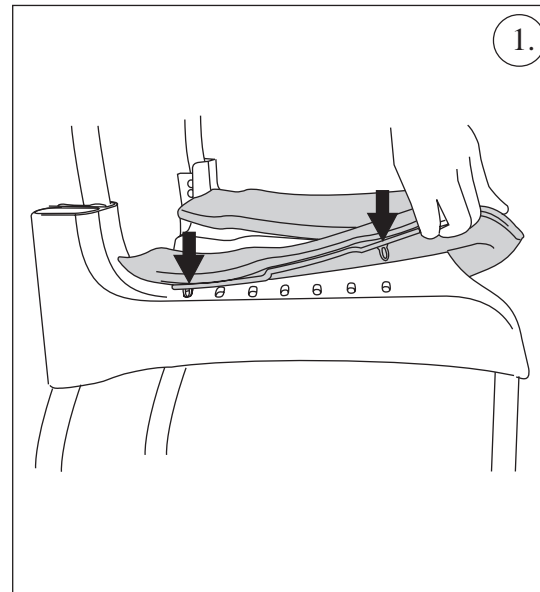
1. Mykt sete kneppes fast på setet.
2. Myk rygg kneppes fast på ryggstøtten.

(Eng) Accessories

1. The soft seat clips onto the seat.
2. The soft back clips onto the backrest.

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